CONCENTRADOR DE OXÍGENO  
**CARE ZY-5AF**

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# CONCENTRADOR DE OXÍGENO

# CARE ZY-5AF

## *Pesa sólo 16 Kg*

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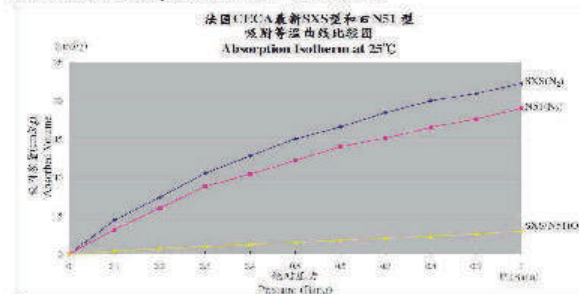
# CONCENTRADOR DE OXÍGENO CARE ZY-5AF

Sieve tank specification sheet

Model	Tank Length (mm)	Sieve brand	Weight of sieves pair	Cycle time	O2 tank pressure / Kpa	O2 pressure after valve / Kpa
SL	270	CECA France	802g	6s	<170	40-55
BL	300	CECA France	1022g	6s	<180	40-60

Molecular Sieve model: CECA SXS (Made in France)

Here below is the absorption isotherm details of SXS sieve.

The diameter of SXS sieve bed is 0.4-0.6mm, it is a high efficiency type sieves which is widely used in PSA medical grade oxygen concentrators by manufacturer from USA, France, Japan and China.

The ordinary method of sieve tank filling is potentially affected by few elements, for example the electrical weighting machine, it has the tolerance rate and will affect the accuracy and balance of sieve beds volume in different tanks.

To avoid such kind of failure rate, we have design a special sieve bed filling machine. We obtain the patent for this fill machine. Here below is the drawing of filling device.

Meanwhile, the sieve tank is also facing the risk of air leakage which will damage the sieve efficiency and wet the chemical sieve bed. To avoid the air leakage, we have 2 patent solutions which are:

1. Three degree tank sealing. (A. Tank seal, B. Valve seal, C. O2 tank seal)
2. Two degree one-way valve sealing (A. Nitrogen one way seal, B. Oxygen outlet one way seal).

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